

## URGENT FIELD SAFETY NOTICE

September 6, 2015

Surgeon Name  
Address  
City, State, Zip

RE: Voluntary Recall of Specific Serial Numbers of ConforMIS Knee Replacement Systems

Dear Dr. X,

ConforMIS has initiated a voluntary recall of specifically identified serial numbers of patient specific knee replacement systems. We initiated this action in response to three recent complaints of moisture on the patient-specific instrumentation. In all three cases, the knee replacement procedures were completed without apparent incident. The Company does not believe that the customized knee implants used in these procedures were themselves affected. While the number of complaints the Company has received is small, and the Company has not received any reports of adverse events related to these complaints to date, the Company initiated this recall voluntarily and is working to resolve the complaints quickly.

Based on an initial assessment, ConforMIS believes that the recalled instrumentation held excess water before undergoing the commonly used ethylene oxide sterilization process and, as a result, may contain small amounts of ethylene glycol residue. Ethylene glycol residue may form when ethylene oxide comes into contact with water. ConforMIS has temporarily suspended its use of the ethylene oxide sterilization process and is working expeditiously to investigate the root cause of the excess moisture and evaluate potential corrective and preventative actions.

The part number, description, serial number, and patient name of units impacted by the voluntary recall are listed below. If any of these kits were not implanted in patients, they must be returned to your ConforMIS representative immediately.

Part Number	Description	Serial Number	Patient Name

As part of our investigation, an independent testing laboratory has determined that no ethylene glycol residue was present on implants tested following sterilization using ethylene oxide. In addition, the independent testing laboratory has determined the risk of ethylene glycol induced toxicity from exposure to the instruments is low, because, in part, ethylene glycol does not represent a practical health hazard from exposure to medical devices for exposures less than 24 hours in duration. Based on the test results to this point, ConforMIS believes no additional monitoring of patients is necessary related to this recall.

Please fill out Attachment A, acknowledging receipt of this letter and confirming the status of the affected units. We have notified the appropriate regulatory agencies and will be providing them with surveillance documentation regarding the effectiveness of the recall communication.

If you have any questions about this notice, please contact your local ConforMIS representative or myself, Amita Shah, at 1-781-345-9164.

Sincerely,

Amita Shah  
Senior Vice President, Quality and Regulatory Affairs  
ConforMIS, Inc.

## ATTACHMENT A – CONFIRMATION OF LETTER RECEIPT & STATUS

Please fill out and return completed form to the ConforMIS Representative or Fax to (781) 345-0147

ConforMIS has provided and I have read the recall notification informing me of the recall of specific serial numbers. The part number, description, serial number, and patient name for affected units are listed below. Regarding patients implanted with recalled product, based on the test results to this point, ConforMIS believes no additional monitoring of patients is necessary related to this recall.

Part Number	Description	Serial Number	Patient Name	Status
				<input type="checkbox"/> Returned to ConforMIS Representative <input type="checkbox"/> Previously Implanted on _____

Surgeon Name: Dr. X

Office Address: Street  
City, TX 75051

Surgeon Signature: \_\_\_\_\_ Date: \_\_\_\_\_

ConforMIS Field Safety Corrective Action

Catalog Number	Description	Serial Number	Hospital Name	Shipped Date	Expiration Date	Ship To Address Line 1	Ship To Address	Ship To City	Ship To State	Ship To Zip Code	Ship To Country
M5722INT0600210	iUNI G2, Left Medial	0349592	Agaplesion Evangelisches Krankenhaus Mittelhessen	08/17/2015	1/31/2016	z.Hd. Herrn Chefarzt Prof. Dr. Jens Kordelle	Paul-Zipp-Strasse 171	Giessen		35398	DE
M5722INT0600220	iUNI G2, Right Medial	0347979	Alexianer Krefeld GmbH	07/27/2015	12/31/2015	-OP-Abteilung-	Diessemer Bruch 81	Krefeld		47805	DE
M5722INT0600210	iUNI G2, Left Medial	0352827	Alexianer Krefeld GmbH	08/04/2015	12/31/2015	-OP-Abteilung-	Diessemer Bruch 81	Krefeld		47805	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354607	Allgemeines Krankenhaus Hagen gGmbH	08/14/2015	1/31/2016	OP-Abteilung Herr Spannenkrebs	Gruenstr. 35	Hagen		58095	DE
M5723INT06000210	iDUO G2, Left Medial	0353143	AMEOS Klinikum Schoenebeck	08/04/2015	12/31/2015	OP-Bereich Operateur: Dirk Rommel	Koethener Strasse 13	Schoenebeck		39218	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353145	AMEOS Klinikum Schoenebeck	08/04/2015	12/31/2015	OP-Bereich Operateur: Dirk Rommel	Koethener Strasse 13	Schoenebeck		39218	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354287	Bezirksklinikum Obermain	08/04/2015	12/31/2015	OP-Abtlg. Orthopaedischer OP fuer Dr. Hellich	Kutzenberg	Ebensfeld		96250	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354383	Bezirksklinikum Obermain	08/04/2015	12/31/2015	OP-Abtlg. Orthopaedischer OP fuer Dr. Hellich	Kutzenberg	Ebensfeld		96250	DE
M5725INT0600010	iTOTAL CR, Left Knee	0351118	BMI The Beaumont Hospital	08/03/2015	12/31/2015	VAT: GB235134980	Attn: Theatres	Bolton	Lancashire	BL6 4LA	GB
M5722INT0600210	iUNI G2, Left Medial	0351616	eHnv - Etablissements Hospitaliers du Nord Vaudois	08/10/2015	12/31/2015	Bloc Operatoire	Mme Baert/Mme Beckers	Yverdon-les-Bains		CH-1400	CH
M5725INT0600010	iTOTAL CR, Left Knee	0353842	eHnv - Etablissements Hospitaliers du Nord Vaudois	08/10/2015	1/31/2016	Bloc Operatoire	Mme Baert/Mme Beckers	Yverdon-les-Bains		CH-1400	CH
M5725INT0600020	iTOTAL CR, Right Knee	0354199	eHnv - Etablissements Hospitaliers du Nord Vaudois	08/17/2015	1/31/2016	Bloc Operatoire	Mme Baert/Mme Beckers	Yverdon-les-Bains		CH-1400	CH
M5725INT0600020	iTOTAL CR, Right Knee	0355061	eHnv - Etablissements Hospitaliers du Nord Vaudois	08/10/2015	1/31/2016	Bloc Operatoire	Mme Baert/Mme Beckers	Yverdon-les-Bains		CH-1400	CH
M5725INT0600010	iTOTAL CR, Left Knee	0354182	EKA Erzgebirgsklinikum Annaberg gGmbH	08/18/2015	1/31/2016	Einkauf/ Materialwirtschaft	Chemnitzer Strasse 15	Annaberg-Buchholz		09456	DE
M5722INT0600220	iUNI G2, Right Medial	0354600	EKA Erzgebirgsklinikum Annaberg gGmbH	08/18/2015	1/31/2016	Einkauf/ Materialwirtschaft	Chemnitzer Strasse 15	Annaberg-Buchholz		09456	DE
M5725INT0600010	iTOTAL CR, Left Knee	0349828	Elblandkliniken Radebeul	08/03/2015	12/31/2015	Chefarzt Dr. Axel Ripp	Klinik fuer Unfallchirurgie + Orthopaedie	Radebeul		01445	DE
TPS-111-1111	iTOTAL PS-IPOLY-IMPLANT KIT-RIGHT	0353150	Elblandkliniken Radebeul	07/24/2015	12/31/2015	Chefarzt Dr. Axel Ripp	Klinik fuer Unfallchirurgie + Orthopaedie	Radebeul		01445	DE
TPS-111-1111	iTOTAL PS-IPOLY-IMPLANT KIT-LEFT	0353544	Elblandkliniken Radebeul	07/24/2015	12/31/2015	Chefarzt Dr. Axel Ripp	Klinik fuer Unfallchirurgie + Orthopaedie	Radebeul		01445	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354067	Elblandkliniken Radebeul	08/04/2015	12/31/2015	Chefarzt Dr. Axel Ripp	Klinik fuer Unfallchirurgie + Orthopaedie	Radebeul		01445	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354070	Elblandkliniken Radebeul	08/04/2015	12/31/2015	Chefarzt Dr. Axel Ripp	Klinik fuer Unfallchirurgie + Orthopaedie	Radebeul		01445	DE
M5725INT0600010	iTOTAL CR, Left Knee	0351132	Elblandkliniken Radebeul	08/17/2015	1/31/2016	Chefarzt Dr. Axel Ripp	Klinik fuer Unfallchirurgie + Orthopaedie	Radebeul		01445	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354066	Elblandkliniken Radebeul	08/18/2015	1/31/2016	Chefarzt Dr. Axel Ripp	Klinik fuer Unfallchirurgie + Orthopaedie	Radebeul		01445	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354068	Elblandkliniken Radebeul	08/18/2015	1/31/2016	Chefarzt Dr. Axel Ripp	Klinik fuer Unfallchirurgie + Orthopaedie	Radebeul		01445	DE
M5725INT0600020	iTOTAL CR, Right Knee	0352859	Ev. Krankenhaus Hattingen gGmbH	08/21/2015	1/31/2016	Zentral-OP Dr. med. Kamminga	Bredenscheider Str. 54	Hattingen		45525	DE
M5722INT0600220	iUNI G2, Right Medial	0349420	Evang. Krankenhaus - Unna	08/04/2015	12/31/2015	Orthopaedie	Dr. J. Heinzkyll	Unna		59423	DE
M5725INT0600010	iTOTAL CR, Left Knee	0353713	Evangelische Elisabeth Klinik	08/11/2015	1/31/2016	OP Abt. Dr. med. Kai Roske	Luetzowstrasse 24 -26	Berlin		10785	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353992	Evangelisches Krankenhaus Weyertal	08/03/2015	12/31/2015	Orthopaedische Gemeinschaftspraxis**	Drs. Heyl und Enderer	Koeln		50939	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354536	Evangelisches Krankenhaus Weyertal	08/17/2015	1/31/2016	Orthopaedische Gemeinschaftspraxis**	Drs. Heyl und Enderer	Koeln		50939	DE
M5722INT0600220	iUNI G2, Right Medial	0354646	Evangelisches Krankenhaus Weyertal	08/11/2015	1/31/2016	Orthopaedische Gemeinschaftspraxis**	Drs. Heyl und Enderer	Koeln		50939	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354688	Evangelisches Krankenhaus Weyertal	08/18/2015	1/31/2016	Orthopaedische Gemeinschaftspraxis**	Drs. Heyl und Enderer	Koeln		50939	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353184	Fairfield Independent Hospital	08/12/2015	1/31/2016	Attn: Theatres	Crank Road	St. Helens	Merseyside	WA11 7RS	GB
M5725INT0600020	iTOTAL CR, Right Knee	0353240	Goring Hall Hospital	08/03/2015	12/31/2015	VAT: 235134980	Bodiam Avenue	Goring-by-Sea	West Sussex	BN12 5AT	GB
M5725INT0600020	iTOTAL CR, Right Knee	0353800	Havelklinik	08/03/2015	12/31/2015	OP-Abteilung	Dr. Wolfgang Muench	Berlin		13595	DE
M5725INT0600020	iTOTAL CR, Right Knee	0351749	HEH - Herzogin Elisabeth Hospital	07/28/2015	12/31/2015	OP-Ltg. Herr Peter Jagdmann	Leipziger Strasse 24	Braunschweig		38124	DE
M5725INT0600010	iTOTAL CR, Left Knee	0352607	HEH - Herzogin Elisabeth Hospital	08/12/2015	12/31/2015	OP-Ltg. Herr Peter Jagdmann	Leipziger Strasse 24	Braunschweig		38124	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353709	HEH - Herzogin Elisabeth Hospital	08/12/2015	12/31/2015	OP-Ltg. Herr Peter Jagdmann	Leipziger Strasse 24	Braunschweig		38124	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353216	HEH - Herzogin Elisabeth Hospital	08/12/2015	12/31/2015	OP-Ltg. Herr Peter Jagdmann	Leipziger Strasse 24	Braunschweig		38124	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354623	HEH - Herzogin Elisabeth Hospital	08/18/2015	1/31/2016	OP-Ltg. Herr Peter Jagdmann	Leipziger Strasse 24	Braunschweig		38124	DE
M5725INT0600020	iTOTAL CR, Right Knee	0355051	HEH - Herzogin Elisabeth Hospital	08/21/2015	1/31/2016	OP-Ltg. Herr Peter Jagdmann	Leipziger Strasse 24	Braunschweig		38124	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353246	Helmut-G.-Walther-Klinikum Lichtenfels GmbH	08/04/2015	12/31/2015	z. Hd. OP-Abteilungsleitung	CA Dr. Mueller	Lichtenfels		96215	DE
M5722INT0600220	iUNI G2, Right Medial	0354561	Helmut-G.-Walther-Klinikum Lichtenfels GmbH	08/07/2015	12/31/2015	z. Hd. OP-Abteilungsleitung	CA Dr. Mueller	Lichtenfels		96215	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353521	Herz-Jesu-Krankenhaus Hiltrup GmbH	07/28/2015	12/31/2015	Abteilung Sekretariat Unfall-, Hand- & Orthopaedische Chirurgie	Dr. med. Friedrich Georg Scherf	Muenster		48165	DE
M5725INT0600010	iTOTAL CR, Left Knee	0353922	Huemmling Krankenhaus Soegel	08/14/2015	1/31/2016	OP / Herr Gerd Mueller	Muehlensstrasse 17	Soegel		49751	DE
M5725INT0600020	iTOTAL CR, Right Knee	0352716	Isar Kliniken GmbH	08/10/2015	12/31/2015	c/o Fr. Dittbrand	Sonnenstrasse 24 - 26	Muenchen		80331	DE
M5722INT0600210	iUNI G2, Left Medial	0353921	Klinik Bad Windsheim	07/27/2015	12/31/2015	OP-Abteilung	Erkenbrechtallee 45	Bad Windsheim		91438	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354535	Klinik Hohe Warte	08/18/2015	1/31/2016	OP-Abteilung	Stellv. Leitung Sr. Melanie	Bayreuth		95445	DE

ConforMIS Field Safety Corrective Action

Catalog Number	Description	Serial Number	Hospital Name	Shipped Date	Expiration Date	Ship To Address Line 1	Ship To Address	Ship To City	Ship To State	Ship To Zip Code	Ship To Country
M5725INT0600020	iTOTAL CR, Right Knee	0354787	Klinik Hohe Warte	08/18/2015	1/31/2016	OP-Abteilung	Stellv. Leitung Sr. Melanie	Bayreuth		95445	DE
M5722INT0600220	iUNI G2, Right Medial	0351754	Klinik Koesching	07/28/2015	12/31/2015	OP-Leitung Orthopaedie	Krankenhausstrasse 19	Koesching		85092	DE
M5722INT0600210	iUNI G2, Left Medial	0354208	Klinik Koesching	08/04/2015	12/31/2015	OP-Leitung Orthopaedie	Krankenhausstrasse 19	Koesching		85092	DE
M5723INT0600220	iDUO G2, Right Medial	0352344	Kliniken Dr. Erler Freigemeinnuetziges Fachkrankenhaus	08/04/2015	12/31/2015	Orthopaedischer OP, OP-Leitung	z.Hd. Sr. Andrea	Nuernberg		90429	DE
M5722INT0600210	iUNI G2, Left Medial	0352999	Kliniken Dr. Erler Freigemeinnuetziges Fachkrankenhaus	07/28/2015	12/31/2015	Orthopaedischer OP, OP-Leitung	z.Hd. Sr. Andrea	Nuernberg		90429	DE
M5722INT0600240	iUNI G2, Right Lateral	0352836	Klinikum Dortmund	08/07/2015	12/31/2015	Ortho-OP z. Hd. Herrn Martin Morher	Beurhausstr. 40	Dortmund		44137	DE
M5725INT0600020	iTOTAL CR, Right Knee	0349077	Klinikum Dortmund	07/27/2015	12/31/2015	Ortho-OP z. Hd. Herrn Martin Morher	Beurhausstr. 40	Dortmund		44137	DE
M5725INT0600020	iTOTAL CR, Right Knee	0349954	Klinikum Dortmund	07/28/2015	12/31/2015	Ortho-OP z. Hd. Herrn Martin Morher	Beurhausstr. 40	Dortmund		44137	DE
M5725INT0600010	iTOTAL CR, Left Knee	0350960	Klinikum Dortmund	07/27/2015	12/31/2015	Ortho-OP z. Hd. Herrn Martin Morher	Beurhausstr. 40	Dortmund		44137	DE
M5725INT0600020	iTOTAL CR, Right Knee	0351376	Klinikum Dortmund	08/04/2015	12/31/2015	Ortho-OP z. Hd. Herrn Martin Morher	Beurhausstr. 40	Dortmund		44137	DE
M5725INT0600010	iTOTAL CR, Left Knee	0352275	Klinikum Dortmund	08/04/2015	12/31/2015	Ortho-OP z. Hd. Herrn Martin Morher	Beurhausstr. 40	Dortmund		44137	DE
M5725INT0600010	iTOTAL CR, Left Knee	0352587	Klinikum Dortmund	08/03/2015	12/31/2015	Ortho-OP z. Hd. Herrn Martin Morher	Beurhausstr. 40	Dortmund		44137	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354290	Klinikum Forchheim	08/01/2015	12/31/2015	Unfallchirurgie	PD Dr. med. Uwe Lehmann	Forchheim		91301	DE
M5722INT0600230	iUNI G2, Left Lateral	0354849	Klinikum Forchheim	08/01/2015	12/31/2015	Unfallchirurgie	PD Dr. med. Uwe Lehmann	Forchheim		91301	DE
M5723INT0600240	iDUO G2, Right Lateral	0354195	Klinikum Frankfurt Hoechst GmbH	08/10/2015	1/31/2016	z.Hd. Herrn Chefarzt Prof. Dr. W. Daecke	Klinik fuer Orthopaedie und Unfallchirurgie	Frankfurt am Main		65929	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353982	Klinikum Lippe Lemgo	08/11/2015	1/31/2016	Dr. Wojciechowski	Zentral-OP	Lemgo		32657	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353213	Klinikum Nuernberg	08/04/2015	12/31/2015	Sued	ZSVA - Sr. Regina	Nuernberg		90471	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354550	Klinikum Rechts der Isar	08/04/2015	12/31/2015	Orthopaedischer OP, Bau 501, 2.Stock	Frau Inge Gindorf	Muenchen		81675	DE
M5725INT0600020	iTOTAL CR, Right Knee	0352084	Klinikum Rechts der Isar	08/14/2015	1/31/2016	Orthopaedischer OP, Bau 501, 2.Stock	Frau Inge Gindorf	Muenchen		81675	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353710	Knappschaftskrankenhaus Puettlingen	07/27/2015	12/31/2015	Herrn Dr. Karsten Kurz	In der Humes 35	Puettlingen		66346	DE
M5723INT0600210	iDUO G2, Left Medial	0353852	Knappschaftskrankenhaus Puettlingen	08/04/2015	12/31/2015	Herrn Dr. Karsten Kurz	In der Humes 35	Puettlingen		66346	DE
M5725INT0600010	iTOTAL CR, Left Knee	0353299	Krankenhaus Maria-Hilf Stadtlöhn GmbH	07/27/2015	12/31/2015	Orthopaedie/Unfallchirurgie MVZ	Dr.med.Ulrich Rebers	Stadtlöhn		48703	DE
M5722INT0600220	iUNI G2, Right Medial	0351218	Krankenhaus Markt Werneck	07/28/2015	12/31/2015	OP-Leitung, Hr. Michael Reinhard	Balthasar-Neumann-Platz 5	Werneck		97440	DE
M5725INT0600020	iTOTAL CR, Right Knee	0352839	Krankenhaus Markt Werneck	07/27/2015	12/31/2015	OP-Leitung, Hr. Michael Reinhard	Balthasar-Neumann-Platz 5	Werneck		97440	DE
M5722INT0600210	iUNI G2, Left Medial	0353210	Krankenhaus Markt Werneck	07/27/2015	12/31/2015	OP-Leitung, Hr. Michael Reinhard	Balthasar-Neumann-Platz 5	Werneck		97440	DE
M5722INT0600220	iUNI G2, Right Medial	0353745	Krankenhaus Markt Werneck	07/27/2015	12/31/2015	OP-Leitung, Hr. Michael Reinhard	Balthasar-Neumann-Platz 5	Werneck		97440	DE
M5722INT0600210	iUNI G2, Left Medial	0354163	Krankenhaus Markt Werneck	08/03/2015	12/31/2015	OP-Leitung, Hr. Michael Reinhard	Balthasar-Neumann-Platz 5	Werneck		97440	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354258	Krankenhaus Markt Werneck	08/17/2015	1/31/2016	OP-Leitung, Hr. Michael Reinhard	Balthasar-Neumann-Platz 5	Werneck		97440	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354398	Krankenhaus Markt Werneck	08/17/2015	1/31/2016	OP-Leitung, Hr. Michael Reinhard	Balthasar-Neumann-Platz 5	Werneck		97440	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354728	Krankenhaus Markt Werneck	08/21/2015	1/31/2016	OP-Leitung, Hr. Michael Reinhard	Balthasar-Neumann-Platz 5	Werneck		97440	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354824	Krankenhaus Markt Werneck	08/19/2015	1/31/2016	OP-Leitung, Hr. Michael Reinhard	Balthasar-Neumann-Platz 5	Werneck		97440	DE
TPS-111-1111-010201	iTOTAL PS-IPOLY-IMPLANT KIT-LEFT	0352550	Krankenhaus Woerth an der Donau	07/27/2015	12/31/2015	OP-Abteilung, z.Hd. Rudolph Rene	Krankenhausstrasse 2	Woerth		93086	DE
M5725INT0600010	iTOTAL CR, Left Knee	0353033	Krankenhaus Woerth an der Donau	08/04/2015	12/31/2015	OP-Abteilung, z.Hd. Rudolph Rene	Krankenhausstrasse 2	Woerth		93086	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354322	Krankenhaus Woerth an der Donau	08/11/2015	1/31/2016	OP-Abteilung, z.Hd. Rudolph Rene	Krankenhausstrasse 2	Woerth		93086	DE
M5725INT0600010	iTOTAL CR, Left Knee	0351991	Kreisklinikum Calw	08/04/2015	12/31/2015		Eduard-Conz-Str. 6	Calw		75365	DE
M5725INT0600020	iTOTAL CR, Right Knee	0352862	Kreisklinikum Calw	08/03/2015	12/31/2015		Eduard-Conz-Str. 6	Calw		75365	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353295	Kreisklinikum Calw	08/04/2015	12/31/2015		Eduard-Conz-Str. 6	Calw		75365	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354682	Kreisklinikum Calw	08/14/2015	1/31/2016		Eduard-Conz-Str. 6	Calw		75365	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353291	London Bridge Hospital	07/28/2015	12/31/2015	VAT: 657159115	Attn:Theatre, 27 Tooley Str.	London		SE1 2PR	GB
M5725INT0600020	iTOTAL CR, Right Knee	0354657	Luisenhospital Aachen	08/14/2015	1/31/2016	OP Abtlg. Sr. Vesna	Sr. Juliane	Aachen		52064	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354187	Marien-Hospital Erwitte	08/18/2015	1/31/2016	z.Hd. Herrn Prof.Dr.med. Thomas Hess	Von -Droste-Strasse 14	Erwitte		59597	DE
M5722INT0600210	iUNI G2, Left Medial	0354507	MIO Sportklinik GbR	08/17/2015	1/31/2016	OP-Bereich	Richard-Lehmann-Str. 21	Leipzig		04275	DE
M5722INT0600220	iUNI G2, Right Medial	0353416	Oberlausitz Kliniken gGmbH	07/27/2015	12/31/2015	Herrn Chefarzt Dr. Marc Naupert	Unfallchirurgie	Bischofswerda		01877	DE
M5725INT0600010	iTOTAL CR, Left Knee	0353804	Oberlausitz Kliniken gGmbH	07/27/2015	12/31/2015	Herrn Chefarzt Dr. Marc Naupert	Unfallchirurgie	Bischofswerda		01877	DE
M5722INT0600210	iUNI G2, Left Medial	0351522	Oberlausitz Kliniken gGmbH	08/12/2015	1/31/2016	Herrn Chefarzt Dr. Marc Naupert	Unfallchirurgie	Bischofswerda		01877	DE
M5722INT0600220	iUNI G2, Right Medial	0352918	Orthopaedische Universitaetsklinik	08/07/2015	12/31/2015	Attn: OP-Abteilung	Koenig-Ludwig-Haus	Wuerzburg		97074	DE
M5723INT0600220	iDUO G2, Right Medial	0353845	Orthopaedische Universitaetsklinik	08/01/2015	12/31/2015	Attn: OP-Abteilung	Koenig-Ludwig-Haus	Wuerzburg		97074	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354444	Paracelsus Kliniken Klinik Osnabrueck	08/01/2015	12/31/2015	Zentral-OP Sr. Silvia	Am Natruper Holz 69	Osnabrueck		49076	DE
M5723INT0600220	iDUO G2, Right Medial	0352431	Princess Margaret Hospital	07/27/2015	12/31/2015	Attn: Theatre	38-42 Osborne Road	Windsor	Berkshire	SL4 3SJ	GB
M5725INT0600010	iTOTAL CR, Left Knee	0354397	Princess Margaret Hospital	08/03/2015	12/31/2015	Attn: Theatre	38-42 Osborne Road	Windsor	Berkshire	SL4 3SJ	GB
M5723INT0600210	iDUO G2, Left Medial	0353859	Princess Margaret Hospital	08/17/2015	1/31/2016	Attn: Theatre	38-42 Osborne Road	Windsor	Berkshire	SL4 3SJ	GB
M5725INT0600010	iTOTAL CR, Left Knee	0354402	Princess Margaret Hospital	08/17/2015	1/31/2016	Attn: Theatre	38-42 Osborne Road	Windsor	Berkshire	SL4 3SJ	GB
M5725INT0600020	iTOTAL CR, Right Knee	0354403	Princess Margaret Hospital	08/17/2015	1/31/2016	Attn: Theatre	38-42 Osborne Road	Windsor	Berkshire	SL4 3SJ	GB
M5725INT0600020	iTOTAL CR, Right Knee	0352778	Regio Kliniken GmbH	08/04/2015	12/31/2015	Regio Kliniken Pinneberg	OP Abt. Sr. Dagmar	Pinneberg		25421	DE
M5725INT0600020	iTOTAL CR, Right Knee	0352469	Regio Kliniken GmbH	08/21/2015	1/31/2016	Regio Kliniken Pinneberg	OP Abt. Sr. Dagmar	Pinneberg		25421	DE
M5725INT0600020	iTOTAL CR, Right Knee	0352470	Regio Kliniken GmbH	08/21/2015	1/31/2016	Regio Kliniken Pinneberg	OP Abt. Sr. Dagmar	Pinneberg		25421	DE

ConforMIS Field Safety Corrective Action

Catalog Number	Description	Serial Number	Hospital Name	Shipped Date	Expiration Date	Ship To Address Line 1	Ship To Address	Ship To City	Ship To State	Ship To Zip Code	Ship To Country
M5725INT0600020	iTOTAL CR, Right Knee	0354458	Regio Kliniken GmbH	08/14/2015	1/31/2016	Regio Kliniken Pinneberg	OP Abt. Sr. Dagmar	Pinneberg		25421	DE
M5722INT0600210	iUNI G2, Left Medial	0353225	Rotkreuzklinikum Sued	08/11/2015	1/31/2016	Zentral-OP. Pfl. Johannes Lochner	PD Dr. Stephan Radke	Muenchen		80634	DE
M5725INT0600020	iTOTAL CR, Right Knee	0352407	Sana Klinikum Berlin - Lichtenberg	07/27/2015	12/31/2015	OP Abteilung - Dr. Ralf Graeser	Fanningersstrasse 32	Berlin		10365	DE
M5725INT0600020	iTOTAL CR, Right Knee	0349709	Sana Klinikum Berlin - Lichtenberg	08/10/2015	1/31/2016	OP Abteilung - Dr. Ralf Graeser	Fanningersstrasse 32	Berlin		10365	DE
M5725INT0600010	iTOTAL CR, Left Knee	0351546	Sankt Josef-Hospital GmbH	08/04/2015	12/31/2015	Abteilung Unfallchirurgie und Orthopaedie	Dr.med.Olaf Lutz Nosseir	Xanten		46509	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353658	Sankt Josef-Hospital GmbH	08/07/2015	12/31/2015	Abteilung Unfallchirurgie und Orthopaedie	Dr.med.Olaf Lutz Nosseir	Xanten		46509	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354733	Sankt Josef-Hospital GmbH	08/14/2015	1/31/2016	Abteilung Unfallchirurgie und Orthopaedie	Dr.med.Olaf Lutz Nosseir	Xanten		46509	DE
M5725INT0600020	iTOTAL CR, Right Knee	0355094	Sankt Josef-Hospital GmbH	08/17/2015	1/31/2016	Abteilung Unfallchirurgie und Orthopaedie	Dr.med.Olaf Lutz Nosseir	Xanten		46509	DE
M5722INT0600210	iUNI G2, Left Medial	0354439	Sportklinik Stuttgart GmbH	08/10/2015	1/31/2016	OP-Abteilung	Taubenheimstr. 8	Stuttgart		70372	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354919	St. Bonifatius-Hospital gGmbH	08/11/2015	1/31/2016	Orthopaedie Dr. med. Holger	Alex Wilhelmstrasse 13	Lingen		49808	DE
M5722INT0600210	iUNI G2, Left Medial	0353714	St. Christophorus Krankenhaus GmbH	07/27/2015	12/31/2015	z. Hd Dr.med. Josef Kubitschek	Am See 1	Werne		59368	DE
M5725INT0600020	iTOTAL CR, Right Knee	0352601	St. Christophorus Krankenhaus GmbH	08/10/2015	1/31/2016	z. Hd Dr.med. Josef Kubitschek	Am See 1	Werne		59368	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354120	St. Christophorus Krankenhaus GmbH	08/18/2015	1/31/2016	z. Hd Dr.med. Josef Kubitschek	Am See 1	Werne		59368	DE
M5722INT0600220	iUNI G2, Right Medial	0351682	St. Elisabeth Krankenhaus Leipzig	08/01/2015	12/31/2015	OP-Bereich, z.Hd. Dr. Kohlschreiber	Biedermannstrasse 84	Leipzig		04277	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354011	St. Elisabeth Krankenhaus Leipzig	07/27/2015	12/31/2015	OP-Bereich, z.Hd. Dr. Kohlschreiber	Biedermannstrasse 84	Leipzig		04277	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354409	St. Elisabeth Krankenhaus Leipzig	08/01/2015	12/31/2015	OP-Bereich, z.Hd. Dr. Kohlschreiber	Biedermannstrasse 84	Leipzig		04277	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353350	St. Franziskus-Krankenhaus Eitorf GmbH	08/04/2015	12/31/2015	OP-Abtlg.	Hospitalstr. 7	Eitorf		53783	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353776	St. Josef-Krankenhaus Linnich	08/18/2015	1/31/2016	Sekretariat Dr. Stephan/Fr. Boehm	Rurdorfer Str. 49	Linnich		52441	DE
M5725INT0600010	iTOTAL CR, Left Knee	0351048	St. Josefs Hospital Cloppenburg Gemeinnuetzige GmbH	07/27/2015	12/31/2015	OP-Abt. - OP-Ltg. Sr. Irmgard	Krankenhausstrasse 13	Cloppenburg		49661	DE
M5725INT0600010	iTOTAL CR, Left Knee	0352403	St. Josefs Hospital Cloppenburg Gemeinnuetzige GmbH	07/28/2015	12/31/2015	OP-Abt. - OP-Ltg. Sr. Irmgard	Krankenhausstrasse 13	Cloppenburg		49661	DE
M5722INT0600220	iUNI G2, Right Medial	0352868	St. Josefs Hospital Cloppenburg Gemeinnuetzige GmbH	08/04/2015	12/31/2015	OP-Abt. - OP-Ltg. Sr. Irmgard	Krankenhausstrasse 13	Cloppenburg		49661	DE
M5725INT0600020	iTOTAL CR, Right Knee	0352876	St. Josefs Hospital Cloppenburg Gemeinnuetzige GmbH	07/28/2015	12/31/2015	OP-Abt. - OP-Ltg. Sr. Irmgard	Krankenhausstrasse 13	Cloppenburg		49661	DE
M5722INT0600210	iUNI G2, Left Medial	0348705	St. Josefs Hospital Cloppenburg Gemeinnuetzige GmbH	08/21/2015	1/31/2016	OP-Abt. - OP-Ltg. Sr. Irmgard	Krankenhausstrasse 13	Cloppenburg		49661	DE
M5722INT0600220	iUNI G2, Right Medial	0352523	St. Josefs Hospital Cloppenburg Gemeinnuetzige GmbH	08/15/2015	1/31/2016	OP-Abt. - OP-Ltg. Sr. Irmgard	Krankenhausstrasse 13	Cloppenburg		49661	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353253	St. Josefs Hospital Cloppenburg Gemeinnuetzige GmbH	08/12/2015	1/31/2016	OP-Abt. - OP-Ltg. Sr. Irmgard	Krankenhausstrasse 13	Cloppenburg		49661	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354984	St. Josefs Krankenhaus gGmbH - Salzkotten	08/14/2015	1/31/2016	Sekretariat fuer Orthopaedie und Unfallchirurgie	z. Hd. Dr.med Birgit Heller	Salzkotten		33154	DE
M5725INT0600010	iTOTAL CR, Left Knee	0355338	St. Josefs Krankenhaus gGmbH - Salzkotten	08/18/2015	1/31/2016	Sekretariat fuer Orthopaedie und Unfallchirurgie	z. Hd. Dr.med Birgit Heller	Salzkotten		33154	DE
M5723INT0600210	iDUO G2, Left Medial	0352932	St. Josef-Stift	08/04/2015	12/31/2015	Dr. F. Horst	Westtor 7	Sendenhorst		48324	DE
M5723INT0600210	iDUO G2, Left Medial	0353049	St. Josef-Stift	08/07/2015	12/31/2015	Dr. F. Horst	Westtor 7	Sendenhorst		48324	DE
M5722INT0600210	iUNI G2, Left Medial	0354292	St. Marienhospital Oberhausen	08/03/2015	12/31/2015	Orthopaedisches Sekretariat	z.Hd. Hr. Dr. Rainer Weber	Oberhausen		46117	DE
M5725INT0600010	iTOTAL CR, Left Knee	0353639	St. Marienhospital Oberhausen	08/18/2015	1/31/2016	Orthopaedisches Sekretariat	z.Hd. Hr. Dr. Rainer Weber	Oberhausen		46117	DE
M5725INT0600020	iTOTAL CR, Right Knee	0355211	St. Marienhospital Oberhausen	08/17/2015	1/31/2016	Orthopaedisches Sekretariat	z.Hd. Hr. Dr. Rainer Weber	Oberhausen		46117	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354177	St. Martinus Krankenhaus	08/18/2015	1/31/2016	OP-Abt./OP-Ltg. Sr. Astrid Reuter	Glabbacher Str. 26	Duesseldorf		40219	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353578	St. Sixtus-Hospital Haltern am See	07/27/2015	12/31/2015	OP-Abteilung, Sr. Bianca/Dr. Asholt	Gartenstr. 2	Haltern am See		45721	DE
M5723INT0600210	iDUO G2, Left Medial	0354930	St. Vincenz Hospital - Essen	08/17/2015	1/31/2016	Dr.med. Feldmann	Von-Bergmann-Str. 2	Essen		45141	DE
M5722INT0600210	iUNI G2, Left Medial	0354509	St. Vinzenz Hospital Dinslaken	08/11/2015	1/31/2016	Abteilung OP	z.Hd. Herrn Pogany	Dinslaken		46535	DE
M5722INT0600220	iUNI G2, Right Medial	0355382	St. Walburga Krankenhaus Meschede GmbH	08/18/2015	1/31/2016	Zentral-OP	Schederweg 12	Meschede		59870	DE
M5723INT0600220	iDUO G2, Right Medial	0353883	Staedtisches Klinikum Brandenburg	08/04/2015	12/31/2015	Klinik fuer Orthopaedie und Unfallchirurgie	OP-Abt. Sr. Gabi	Brandenburg an der Havel		14770	DE
M5723INT0600220	iDUO G2, Right Medial	0353884	Staedtisches Klinikum Brandenburg	08/04/2015	12/31/2015	Klinik fuer Orthopaedie und Unfallchirurgie	OP-Abt. Sr. Gabi	Brandenburg an der Havel		14770	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354071	Staedtisches Klinikum Brandenburg	08/07/2015	12/31/2015	Klinik fuer Orthopaedie und Unfallchirurgie	OP-Abt. Sr. Gabi	Brandenburg an der Havel		14770	DE
M5723INT0600210	iDUO G2, Left Medial	0353621	Staedtisches Klinikum Brandenburg	08/07/2015	1/31/2016	Klinik fuer Orthopaedie und Unfallchirurgie	OP-Abt. Sr. Gabi	Brandenburg an der Havel		14770	DE
M5725INT0600020	iTOTAL CR, Right Knee	0354933	Staedtisches Klinikum Brandenburg	08/11/2015	1/31/2016	Klinik fuer Orthopaedie und Unfallchirurgie	OP-Abt. Sr. Gabi	Brandenburg an der Havel		14770	DE
M5723INT0600210	iDUO G2, Left Medial	0354666	Staedtisches Krankenhaus Nettetal GmbH	08/24/2015	1/31/2016	Sekretariat ZAE	z.Hd. Frau Petra Navis	Nettetal		41334	DE
M5722INT0600220	iUNI G2, Right Medial	0353077	Staedtisches Krankenhaus Nettetal GmbH	08/11/2015	1/31/2016	Sekretariat ZAE	z.Hd. Frau Petra Navis	Nettetal		41334	DE
M5722INT0600220	iUNI G2, Right Medial	0354141	Staedtisches Krankenhaus Nettetal GmbH	08/17/2015	1/31/2016	Sekretariat ZAE	z.Hd. Frau Petra Navis	Nettetal		41334	DE
M5722INT0600220	iUNI G2, Right Medial	0354421	The London Clinic	08/17/2015	1/31/2016	VAT: 494754204	Attn: Main Theatre	London		W1G 6BW	GB
M5722INT0600220	iUNI G2, Right Medial	0351856	Themistocles Gluck Hospital	08/04/2015	12/31/2015	OP-Abt./Herr Axel Sand	c/o Dr. Potrett	Ratingen		40882	DE
TPS-111-1111	iTOTAL PS-IPLY-IMPLANT KIT-RIGHT	0353042	Universitaetsklinikum Halle (Saale)	08/04/2015	12/31/2015	Materialwirtschaft	Ernst-Grube Str. 40	Halle (Saale)		06120	DE
M5725INT0600010	iTOTAL CR, Left Knee	0354239	Universitaetsklinikum Leipzig AoER	08/22/2015	1/31/2016	Orthopaedische Klinik und Poliklinik	OP-Bereich z.Hd. OA Dr. Prielzel	Leipzig		04103	DE
M5723INT0600220	iDUO G2, Right Medial	0354625	Universitaetsspital Basel	08/17/2015	1/31/2016	PD Dr. med. Pagenstert Geert, Stv. Chefarzt	Leitender Arzt Orthopaedie	Basel		4031	CH

**ConforMIS Field Safety Corrective Action**

Catalog Number	Description	Serial Number	Hospital Name	Shipped Date	Expiration Date	Ship To Address Line 1	Ship To Address	Ship To City	Ship To State	Ship To Zip Code	Ship To Country
M5725INT0600020	iTOTAL CR, Right Knee	0354386	Vitos Orthopaedische Klinik Kassel gemeinnuetzige GmbH	08/10/2015	12/31/2015	OP-Abteilung	OP-Ltq. Sr. Gaby	Kassel		34131	DE
M5725INT0600020	iTOTAL CR, Right Knee	0351890	Vivantes Humboldt Klinikum	08/17/2015	1/31/2016	OP - Abteilung Dr. Eller/Kellermann	Am Nordgraben 2	Berlin		13509	DE
M5725INT0600020	iTOTAL CR, Right Knee	0353874	Vivantes Humboldt Klinikum	08/14/2015	1/31/2016	OP - Abteilung Dr. Eller/Kellermann	Am Nordgraben 2	Berlin		13509	DE