Luzernstrasse 21 4528 Zuchwil Switzerland Tel. +41 32 720 40 60 Fax +41 32 720 40 61 http://www.depuysynthes.com



To the ATTENTION of: Operating Room Manager

19 June 2015

URGENT NOTICE: MEDICAL DEVICE RECALL – R20131567 Pull Reduction Device for Percutaneous Drill Guide Ø 4.3mm

Part Description, Part and Lot Numbers

Part Description	Part Number	Lot Numbers						
	03.120.023	1825699	3066995	3139480	3207364	3224275	3764900	
		1825946	3066996	3139483	3218030	3224581	3764900	
Pull Reduction Device for Percutaneous Drill Guide Ø 4.3mm		1959344	3066996	3139485	3218030	3224582	3775645	
		1959346	3066997	3159152	3218068	3224582	3775645	
		3052800	3066997	3167363	3223544	3224583	7502003	
		3052800	3066997	3172147	3223544	3233445	7555756	
		3052800	3092693	3201143	3223545	3233445	7555756	
		3052800	3092737	3201143	3223545	3288202	7625634	
		3052800	3139434	3201143	3224270	3288206	7625634	
		3066995	3139434	3201143	3224271	3326906	8091016	

Dear Sir/Madam,

Synthes GmbH is initiating a voluntary medical device recall of certain lots of the Pull Reduction Device listed above. The Pull Reduction Device for Percutaneous Drill Guide Ø 4.3 mm is placed through the drill guide and plate holes to pull or push bone fragments relative to the plate. This instrument can be used for: Minor Varus-valgus adjustments (approximately 2° - 4°), Translational adjustments, Stabilization of plate bone orientation during insertion of the first screws, alignment of segmental fragments and pre-drilling dense or thick cortical bone before placing a 5.0 mm locking screw.

Our records indicate that you may have inventory that is impacted by this recall.

Reason for the Recall:

The affected parts and lots of the Pull Reduction Devices may have been manufactured to an incorrect hardness specification which could result in intraoperative breakage.



Potential hazard:

In the event of an intraoperative breakage a surgical delay could occur. Breakage of the instrument into fragments during active use could lead to a significant prolongation of surgery time as the surgeon attempts to retrieve the pieces including use radiographic imaging to determine location and size of fragment. Efforts at retrieving fragments may also require additional intraoperative maneuvers e.g. supplementary incisions that were not previously intended, leading to soft tissue damage. If the patient is sensitive to retained non-implant grade material of the broken instrument, he or she may develop an inflammatory response to the fragments that were left in the bone leading to adverse tissue reaction.

Customer immediate actions:

- Immediately identify and quarantine all unused products listed above in a manner that ensures the affected products will not be used.
- Review, complete, sign and return the attached reply form on page 3 of this letter to your local DePuy Synthes sales organization in accordance with the directions on the form within 5 business days of receipt of this notification.
- Return any affected product as soon as possible, but within 30 business days. A credit note will be issued for the returned items.
- 4. Forward this notice to anyone in your facility that needs to be informed.
- 5. If any of the affected products have been forwarded to another facility, contact that facility to arrange return.
- Maintain awareness of this notice until all products listed below have been returned to DePuy Synthes.
- Keep a copy of this notice.

We apologize for any inconvenience that this product recall may create and appreciate your cooperation with our request. Should you have any inquiries please do not hesitate to contact your DePuy Synthes sales consultant.

Thank you for your attention and cooperation.

Synthes GmbH

Pierre van Iwaarden Field Action Manager Charles Goldberg

Worldwide Director Complaint Management

Cc:



Account Name:			
Account Name.			

MEDICAL DEVICE RECALL - R20131567 Pull Reduction Device for Percutaneous Drill Guide Ø4.3mm

Verification Section

Part Description	Part Number			Lot Nu	Lot Numbers				
Pull Reduction Device for	03.120.023	1825699	3066995	3139480	3207364	3224275	3764900		
		1825946	3066996	3139483	3218030	3224581	376490		
Pull Reduction Device for Percutaneous Drill Guide		1959344	3066996	3139485	3218030	3224582	377564		
		1959346	3066997	3159152	3218068	3224582	377564		
		3052800	3066997	3167363	3223544	3224583	750200		
		3052800	3066997	3172147	3223544	3233445	755575		
		3052800	3092693	3201143	3223545	3233445	755575		
		3052800	3092737	3201143	3223545	3288202	762563		
		3052800	3139434	3201143	3224270	3288206	762563		
		3066995	3139434	3201143	3224271	3233445 3233445 3288202	809101		

	3032800	3092737	3201143	3223545	3288202	162
	3052800	3139434	3201143	3224270	3288206	762
	3066995	3139434	3201143	3224271	3326906	809
We have located the identified	d product in st	ock; return	ed quantit	y is docun	nented bel	ow.
We acknowledge receipt of th stock; returned quantity is zer		, but do no	t have any	/ identified	product in	Ì
RETURNED DEVICES (including q	uantity):					
						_
-						
Name/Title (please print):			1000			
Address:						
Phone Number:		1100				
Signature and Date:						

Please complete and return this page your local DePuy Synthes sales organization.

Note: If the Verification Section is answered on behalf of more than one facility and/or individual, please clearly indicate the name and address of the facility and/or individual on this page of the notification.