



URGENT: FIELD SAFETY NOTICE – MMS-21-3991

**BD Alaris™ Pump Module Model 8100
Pump Module Door Assembly Replacement Kits**

March 3, 2021

Dear Valued BD Alaris™ System Customer:
Director of Biomedical Engineering
Director of Nursing
Director of Risk Management

BD is initiating an advisory Field Safety Notice for the BD Alaris™ Pump Module Model 8100 due to potential lifting of the keypad, which could result in one or more keys becoming unresponsive or stuck (i.e., constantly pressed state) due to fluid ingress. This issue may potentially result in a delay to the start of infusion or an interruption of infusion. BD has identified this unique supplier manufacturing issue for Alaris™ System pump modules manufactured between January 23, 2019 to December 5, 2019.

Please note, this Field Safety Notice is separate from the BD Alaris Pump Module 8100 keypad Field Safety Notice dated August 4, 2020.

If you had previously remediated devices affected by the August 4, 2020 Pump Module Keypad Field Safety Notice, those devices will not need to be remediated a second time, as a result of this Field Safety Notice. Please be advised your action is still needed to remediate the devices affected by this Field Safety Notice.

Affected Products:

- BD Alaris™ Pump Module Model 8100 manufactured January 23, 2019 to December 5, 2019.
- Pump Module Door Assembly Replacement Kits labeled with a date range of January 15, 2019 to November 14, 2019. Affected part numbers: 49000239; 49000346; 49000438; 49000439.

See *Attachment A* for a list of affected Alaris™ Pump Module 8100 serial numbers. If applicable, *Attachment A* will also include the quantity of replacement kits shipped.


Issue:

The affected BD Alaris™ Pump Module keypads may exhibit keypad lifting, which could result in one or more keys becoming unresponsive or stuck (i.e., constantly pressed state) due to fluid ingress. Please see Figure 1 for an example of a keypad with no lifting compared to a keypad exhibiting lifting.

- If the keys become unresponsive, the module will continue to infuse as programmed. The unresponsive keypad on the module will not trigger an alarm on the PC unit, and all programming changes must be made on the PC unit.
- If a key gets stuck, the Alaris™ PC unit will alarm and the module will exhibit a Channel Error on the scrolling marquee, which will result in an interruption of infusion.



Figure 1. Alaris™ Pump Module and BD Alaris™ Pump Module demonstrating keypad without lifting compared to a keypad exhibiting lifting.

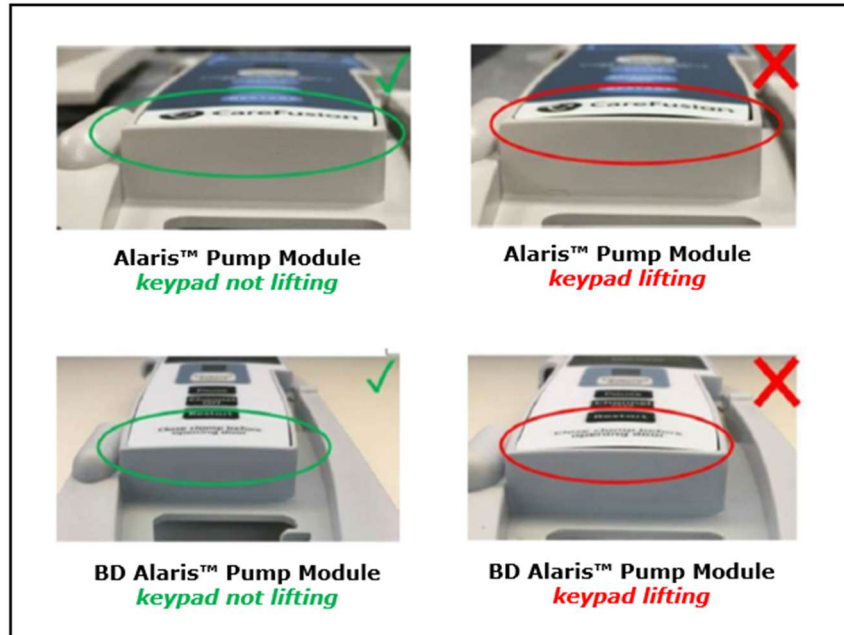

Potential Risks:

An unresponsive keypad could result in a delay to the start of an infusion or inability to titrate medication. If editing of programmed settings is critical, it may be necessary to interrupt and restart the infusion using a different Pump Module.

A stuck key on the Alaris™ Pump Module keypad could result in a delay to the start of an infusion or interruption of infusion.

High-risk patient populations who are receiving life-sustaining infusions are at the greatest risk of harm. For these patients, delay or interruption of an infusion can cause serious injury or death. BD has assessed the risk of this issue and determined that the Alaris™ Pump Modules can still be used until the affected keypads are replaced.

BD has received no reports of permanent injury or death that are potentially related to this issue.

Actions for Clinical Users:

Clinicians should remove the pump from service and send to Biomedical Engineering if the Pump Module keypad shows signs of lifting, becomes unresponsive, or stuck. If a critical medication is being administered, continue the infusion until it is safe to replace the Pump Module. In an urgent situation, clinicians can close the roller clamp on the IV administration set to stop an infusion.

Actions for Cleaning Personnel: Follow the cleaning instructions provided in the current Directions for Use to minimize the potential for fluid ingress during cleaning.



- Do not use a cloth that drips. Be sure to wring out the cleaning cloth to squeeze out excess fluid.
- Do not spray fluids directly onto the device.

If Pump Module keypad lifting or stuck or unresponsive keys are observed, remove the pump from service and send to Biomedical Engineering.

Actions for Biomedical Engineering:

1) Remediation of Pump Module Door Assembly Replacement Kits

All Pump Module Door Assembly Replacement Kits dated from January 15, 2019 to November 14, 2019, should be disposed per facility guidelines. Contact BD to order replacement Pump Module keypad kits at no charge. An example of how to identify the date on the keypad is provided below:



2) Remediation of Pump Module Keypads

Please refer to *Attachment A* for a list of affected serial numbers or visit bd.com/mms-21-3991

3) Customer Reply Form

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
Please promptly complete and return the enclosed Customer Reply Form to acknowledge receipt of this notification and the Field Safety Notice instructions provided in this letter and send it to DL-EMA-INF-Tech-Support@bd.com

Actions by BD:

The root cause was identified as a supplier manufacturing process issue and has been resolved. Corrective actions have been validated and implemented. Any Alaris™ Pump Module keypads on the affected units will be replaced at no charge. BD local representative will coordinate the replacement process.

BD's actions are guided by our commitment to patient safety and minimizing disruption of patient care. We regret the inconvenience that may result from this Field Safety Notice and are committed to achieving the highest levels of customer satisfaction and serving your infusion product needs.

Sincerely,



Jeff Fecho
Vice President, Quality Management
Medication Management Solutions



Idal Beer, MD
Vice President of Medical Affairs
Medication Management Solutions



Customer Acknowledgement Form – MMS-21-3991

BD Alaris™ Pump Module Model 8100 Pump Module Door Assembly Replacement Kits

Please read in conjunction with Product Notification MMS-21-3991 and return the completed and signed form as soon as possible or **no later than March 31st, 2021**, to DL-EMA-INF-Tech-Support@bd.com

By completing the information below you confirm you have read, understood, and distributed the contents of this Product Notification accordingly.

Name of Company	
Email Address	
Telephone Number	
Name	
Signature	
Date	

Serial#	Material	Manufacture Date
15527072	8100TDXEN933	03/22/2019
15527083	8100TDXEN933	03/22/2019
15527088	8100TDXEN933	03/22/2019
15527089	8100TDXEN933	03/22/2019
15527093	8100TDXEN933	03/22/2019
15527094	8100TDXEN933	03/22/2019
15527096	8100TDXEN933	03/22/2019
15527099	8100TDXEN933	03/22/2019
15527100	8100TDXEN933	03/22/2019
15527117	8100TDXEN933	03/22/2019
15527118	8100TDXEN933	03/22/2019
15527120	8100TDXEN933	03/22/2019
15527122	8100TDXEN933	03/22/2019
15527123	8100TDXEN933	03/22/2019
15527131	8100TDXEN933	03/22/2019
15527132	8100TDXEN933	03/22/2019
15527133	8100TDXEN933	03/22/2019
15527134	8100TDXEN933	03/22/2019
15527135	8100TDXEN933	03/22/2019
15527147	8100TDXEN933	03/22/2019
15527148	8100TDXEN933	03/22/2019
15527150	8100TDXEN933	03/22/2019
15527151	8100TDXEN933	03/22/2019
15527162	8100TDXEN933	03/22/2019
15527164	8100TDXEN933	03/22/2019
15527165	8100TDXEN933	03/22/2019
15527182	8100TDXEN933	03/22/2019
15527183	8100TDXEN933	03/22/2019
15527188	8100TDXEN933	03/22/2019
15527189	8100TDXEN933	03/22/2019
15527190	8100TDXEN933	03/22/2019
15527191	8100TDXEN933	03/22/2019
15527192	8100TDXEN933	03/22/2019
15527222	8100TDXEN933	03/22/2019
15527226	8100TDXEN933	03/22/2019
15527229	8100TDXEN933	03/22/2019
15527232	8100TDXEN933	03/22/2019
15527234	8100TDXEN933	03/22/2019
15527237	8100TDXEN933	03/22/2019
15527240	8100TDXEN933	03/22/2019
15527242	8100TDXEN933	03/22/2019
15527244	8100TDXEN933	03/22/2019
15527247	8100TDXEN933	03/22/2019
15527250	8100TDXEN933	03/22/2019
15527252	8100TDXEN933	03/22/2019
15527255	8100TDXEN933	03/22/2019
15527257	8100TDXEN933	03/22/2019
15527261	8100TDXEN933	03/22/2019
15527263	8100TDXEN933	03/22/2019
15527265	8100TDXEN933	03/22/2019
15527269	8100TDXEN933	03/22/2019
15527271	8100TDXEN933	03/22/2019
15527274	8100TDXEN933	03/22/2019
15527277	8100TDXEN933	03/22/2019

15527278 8100TDXEN933 03/22/2019
15527282 8100TDXEN933 03/22/2019
15527286 8100TDXEN933 03/22/2019
15527289 8100TDXEN933 03/22/2019
15527292 8100TDXEN933 03/22/2019
15527295 8100TDXEN933 03/22/2019
15527297 8100TDXEN933 03/22/2019
15527298 8100TDXEN933 03/22/2019
15527300 8100TDXEN933 03/22/2019
15527316 8100TDXEN933 03/22/2019
15527324 8100TDXEN933 03/22/2019
15527330 8100TDXEN933 03/22/2019
15527336 8100TDXEN933 03/22/2019
15527340 8100TDXEN933 03/22/2019
15527345 8100TDXEN933 03/22/2019
15527349 8100TDXEN933 03/22/2019
15527359 8100TDXEN933 03/22/2019
15527363 8100TDXEN933 03/22/2019
15527369 8100TDXEN933 03/22/2019
15527377 8100TDXEN933 03/22/2019
15527380 8100TDXEN933 03/22/2019
15527385 8100TDXEN933 03/22/2019
15527388 8100TDXEN933 03/22/2019
15527393 8100TDXEN933 03/22/2019
15527401 8100TDXEN933 03/22/2019
15527406 8100TDXEN933 03/22/2019
15527409 8100TDXEN933 03/22/2019
15527416 8100TDXEN933 03/22/2019
15527419 8100TDXEN933 03/22/2019
15527424 8100TDXEN933 03/22/2019
15527431 8100TDXEN933 03/22/2019
15527435 8100TDXEN933 03/22/2019
15527443 8100TDXEN933 03/22/2019
15527448 8100TDXEN933 03/22/2019
15527449 8100TDXEN933 03/22/2019
15527454 8100TDXEN933 03/22/2019
15527458 8100TDXEN933 03/22/2019
15527459 8100TDXEN933 03/22/2019
15527466 8100TDXEN933 03/22/2019
15527470 8100TDXEN933 03/22/2019
15527476 8100TDXEN933 03/22/2019
15527479 8100TDXEN933 03/22/2019
15527482 8100TDXEN933 03/22/2019
15527485 8100TDXEN933 03/22/2019
15527488 8100TDXEN933 03/22/2019
15527490 8100TDXEN933 03/22/2019
15527494 8100TDXEN933 03/22/2019
15527514 8100TDXEN933 03/22/2019
15527518 8100TDXEN933 03/22/2019
15533361 8100TDXEN933 03/28/2019
15533380 8100TDXEN933 03/28/2019
15533388 8100TDXEN933 03/28/2019
15533431 8100TDXEN933 03/28/2019
15533516 8100TDXEN933 03/28/2019
15533520 8100TDXEN933 03/28/2019

15533557 8100TDXEN933 03/28/2019
15533562 8100TDXEN933 03/28/2019
15533574 8100TDXEN933 03/28/2019
15533583 8100TDXEN933 03/28/2019
15533592 8100TDXEN933 03/28/2019
15533613 8100TDXEN933 03/28/2019
15533616 8100TDXEN933 03/28/2019
15533619 8100TDXEN933 03/28/2019
15546990 8100TDXEN933 04/11/2019
15546991 8100TDXEN933 04/11/2019
15546993 8100TDXEN933 04/11/2019
15546994 8100TDXEN933 04/11/2019
15546995 8100TDXEN933 04/11/2019
15546996 8100TDXEN933 04/11/2019
15546997 8100TDXEN933 04/11/2019
15546999 8100TDXEN933 04/11/2019
15547001 8100TDXEN933 04/11/2019
15547002 8100TDXEN933 04/11/2019
15547003 8100TDXEN933 04/11/2019
15547004 8100TDXEN933 04/11/2019
15547006 8100TDXEN933 04/11/2019
15547007 8100TDXEN933 04/11/2019
15547009 8100TDXEN933 04/11/2019
15547010 8100TDXEN933 04/11/2019
15547013 8100TDXEN933 04/11/2019
15547014 8100TDXEN933 04/11/2019
15547015 8100TDXEN933 04/11/2019
15547017 8100TDXEN933 04/11/2019
15547019 8100TDXEN933 04/11/2019
15547021 8100TDXEN933 04/11/2019
15547022 8100TDXEN933 04/11/2019
15547023 8100TDXEN933 04/11/2019
15547024 8100TDXEN933 04/11/2019
15547026 8100TDXEN933 04/11/2019
15547027 8100TDXEN933 04/11/2019
15547029 8100TDXEN933 04/11/2019
15547030 8100TDXEN933 04/11/2019
15547031 8100TDXEN933 04/11/2019
15547033 8100TDXEN933 04/11/2019
15547034 8100TDXEN933 04/11/2019
15547035 8100TDXEN933 04/11/2019
15547036 8100TDXEN933 04/11/2019
15547038 8100TDXEN933 04/11/2019
15547039 8100TDXEN933 04/11/2019
15547040 8100TDXEN933 04/11/2019
15547041 8100TDXEN933 04/11/2019
15547042 8100TDXEN933 04/11/2019
15547043 8100TDXEN933 04/11/2019
15547047 8100TDXEN933 04/11/2019
15547049 8100TDXEN933 04/11/2019
15547051 8100TDXEN933 04/11/2019
15547053 8100TDXEN933 04/11/2019
15547057 8100TDXEN933 04/11/2019
15547059 8100TDXEN933 04/11/2019
15547060 8100TDXEN933 04/11/2019

15547061 8100TDXEN933 04/11/2019
15547063 8100TDXEN933 04/11/2019
15547067 8100TDXEN933 04/11/2019
15547122 8100TDXEN933 04/11/2019
15547123 8100TDXEN933 04/11/2019
15547124 8100TDXEN933 04/11/2019
15547125 8100TDXEN933 04/11/2019
15547127 8100TDXEN933 04/11/2019
15547128 8100TDXEN933 04/11/2019
15547129 8100TDXEN933 04/11/2019
15547130 8100TDXEN933 04/11/2019
15547135 8100TDXEN933 04/11/2019
15547136 8100TDXEN933 04/11/2019
15547139 8100TDXEN933 04/11/2019
15547140 8100TDXEN933 04/11/2019
15547142 8100TDXEN933 04/11/2019
15547145 8100TDXEN933 04/11/2019
15547147 8100TDXEN933 04/11/2019
15547151 8100TDXEN933 04/11/2019
15547153 8100TDXEN933 04/11/2019
15547154 8100TDXEN933 04/11/2019
15547156 8100TDXEN933 04/11/2019
15547160 8100TDXEN933 04/11/2019
15547163 8100TDXEN933 04/11/2019
15547166 8100TDXEN933 04/11/2019
15547169 8100TDXEN933 04/11/2019
15547170 8100TDXEN933 04/11/2019
15547172 8100TDXEN933 04/11/2019
15547173 8100TDXEN933 04/11/2019
15547175 8100TDXEN933 04/11/2019
15547178 8100TDXEN933 04/11/2019
15547181 8100TDXEN933 04/11/2019
15547185 8100TDXEN933 04/11/2019
15547187 8100TDXEN933 04/11/2019
15547191 8100TDXEN933 04/11/2019
15547195 8100TDXEN933 04/11/2019
15547197 8100TDXEN933 04/11/2019
15547198 8100TDXEN933 04/11/2019
15547199 8100TDXEN933 04/11/2019
15547201 8100TDXEN933 04/11/2019
15547205 8100TDXEN933 04/11/2019
15547206 8100TDXEN933 04/11/2019
15547207 8100TDXEN933 04/11/2019
15547208 8100TDXEN933 04/11/2019
15547212 8100TDXEN933 04/11/2019
15547214 8100TDXEN933 04/11/2019
15547215 8100TDXEN933 04/11/2019
15547217 8100TDXEN933 04/11/2019
15547218 8100TDXEN933 04/11/2019
15547222 8100TDXEN933 04/11/2019
15547223 8100TDXEN933 04/11/2019
15547225 8100TDXEN933 04/11/2019
15547229 8100TDXEN933 04/11/2019
15547231 8100TDXEN933 04/11/2019
15547233 8100TDXEN933 04/11/2019

15547234 8100TDXEN933 04/11/2019
15547237 8100TDXEN933 04/11/2019
15547238 8100TDXEN933 04/11/2019
15547239 8100TDXEN933 04/11/2019
15547240 8100TDXEN933 04/11/2019
15547241 8100TDXEN933 04/11/2019
15547242 8100TDXEN933 04/11/2019
15547243 8100TDXEN933 04/11/2019
15547244 8100TDXEN933 04/11/2019
15547245 8100TDXEN933 04/11/2019
15547246 8100TDXEN933 04/11/2019
15547247 8100TDXEN933 04/11/2019
15547248 8100TDXEN933 04/11/2019
15547265 8100TDXEN933 04/11/2019
15547267 8100TDXEN933 04/11/2019
15547270 8100TDXEN933 04/11/2019
15547271 8100TDXEN933 04/11/2019
15547273 8100TDXEN933 04/11/2019
15547276 8100TDXEN933 04/11/2019
15547277 8100TDXEN933 04/11/2019
15547278 8100TDXEN933 04/11/2019
15547281 8100TDXEN933 04/11/2019
15547282 8100TDXEN933 04/11/2019
15547285 8100TDXEN933 04/11/2019
15547286 8100TDXEN933 04/11/2019
15547290 8100TDXEN933 04/11/2019
15547291 8100TDXEN933 04/11/2019
15547295 8100TDXEN933 04/11/2019
15547298 8100TDXEN933 04/11/2019
15547299 8100TDXEN933 04/11/2019
15547303 8100TDXEN933 04/11/2019
15547306 8100TDXEN933 04/11/2019
15547307 8100TDXEN933 04/11/2019
15547308 8100TDXEN933 04/11/2019
15547311 8100TDXEN933 04/11/2019
15547313 8100TDXEN933 04/11/2019
15547316 8100TDXEN933 04/11/2019
15547318 8100TDXEN933 04/11/2019
15547319 8100TDXEN933 04/11/2019
15547323 8100TDXEN933 04/11/2019
15547326 8100TDXEN933 04/11/2019
15547328 8100TDXEN933 04/11/2019
15547331 8100TDXEN933 04/11/2019
15547332 8100TDXEN933 04/11/2019
15547335 8100TDXEN933 04/11/2019
15547338 8100TDXEN933 04/11/2019
15547340 8100TDXEN933 04/11/2019
15547341 8100TDXEN933 04/11/2019
15547346 8100TDXEN933 04/11/2019
15547349 8100TDXEN933 04/11/2019
15547351 8100TDXEN933 04/11/2019
15547353 8100TDXEN933 04/11/2019
15547355 8100TDXEN933 04/11/2019
15547357 8100TDXEN933 04/11/2019
15547360 8100TDXEN933 04/11/2019

15547361 8100TDXEN933 04/11/2019
15547362 8100TDXEN933 04/11/2019
15547367 8100TDXEN933 04/11/2019
15547368 8100TDXEN933 04/11/2019
15547369 8100TDXEN933 04/11/2019
15547371 8100TDXEN933 04/11/2019
15547375 8100TDXEN933 04/11/2019
15547380 8100TDXEN933 04/11/2019
15547383 8100TDXEN933 04/11/2019
15547388 8100TDXEN933 04/11/2019
15547389 8100TDXEN933 04/11/2019
15547391 8100TDXEN933 04/11/2019
15547392 8100TDXEN933 04/11/2019
15547395 8100TDXEN933 04/11/2019
15547398 8100TDXEN933 04/11/2019
15547399 8100TDXEN933 04/11/2019
15547418 8100TDXEN933 04/11/2019
15547419 8100TDXEN933 04/11/2019
15547422 8100TDXEN933 04/11/2019
15547423 8100TDXEN933 04/11/2019
15547426 8100TDXEN933 04/11/2019
15547427 8100TDXEN933 04/11/2019
15547431 8100TDXEN933 04/11/2019
15547432 8100TDXEN933 04/11/2019
15547434 8100TDXEN933 04/11/2019
15547436 8100TDXEN933 04/11/2019
15547438 8100TDXEN933 04/11/2019
15547441 8100TDXEN933 04/11/2019
15547442 8100TDXEN933 04/11/2019
15547444 8100TDXEN933 04/11/2019
15547446 8100TDXEN933 04/11/2019
15547447 8100TDXEN933 04/11/2019
15547452 8100TDXEN933 04/11/2019
15547458 8100TDXEN933 04/11/2019
15547460 8100TDXEN933 04/11/2019
15547461 8100TDXEN933 04/11/2019
15547464 8100TDXEN933 04/11/2019
15547468 8100TDXEN933 04/11/2019
15547469 8100TDXEN933 04/11/2019
15547472 8100TDXEN933 04/11/2019
15547473 8100TDXEN933 04/11/2019
15547476 8100TDXEN933 04/11/2019
15547480 8100TDXEN933 04/11/2019
15547481 8100TDXEN933 04/11/2019
15547483 8100TDXEN933 04/11/2019
15547484 8100TDXEN933 04/11/2019
15547488 8100TDXEN933 04/11/2019
15547491 8100TDXEN933 04/11/2019
15547494 8100TDXEN933 04/11/2019
15547495 8100TDXEN933 04/11/2019
15547498 8100TDXEN933 04/11/2019
15547500 8100TDXEN933 04/11/2019
15547501 8100TDXEN933 04/11/2019
15547503 8100TDXEN933 04/11/2019
15547505 8100TDXEN933 04/11/2019

15547507 8100TDXEN933 04/11/2019
15547508 8100TDXEN933 04/11/2019
15547509 8100TDXEN933 04/11/2019
15547510 8100TDXEN933 04/11/2019
15547511 8100TDXEN933 04/11/2019
15547512 8100TDXEN933 04/11/2019
15547522 8100TDXEN933 04/11/2019
15547524 8100TDXEN933 04/11/2019
15547533 8100TDXEN933 04/11/2019
15547534 8100TDXEN933 04/11/2019
15547535 8100TDXEN933 04/11/2019
15547539 8100TDXEN933 04/11/2019
15547540 8100TDXEN933 04/11/2019
15547542 8100TDXEN933 04/11/2019
15547543 8100TDXEN933 04/11/2019
15547545 8100TDXEN933 04/11/2019
15547548 8100TDXEN933 04/11/2019
15547549 8100TDXEN933 04/11/2019
15547551 8100TDXEN933 04/11/2019
15547552 8100TDXEN933 04/11/2019
15547557 8100TDXEN933 04/11/2019
15547558 8100TDXEN933 04/11/2019
15547563 8100TDXEN933 04/11/2019
15547565 8100TDXEN933 04/11/2019
15547567 8100TDXEN933 04/11/2019
15547568 8100TDXEN933 04/11/2019
15547572 8100TDXEN933 04/11/2019
15547576 8100TDXEN933 04/11/2019
15547577 8100TDXEN933 04/11/2019
15547580 8100TDXEN933 04/11/2019
15547583 8100TDXEN933 04/11/2019
15547584 8100TDXEN933 04/11/2019
15547586 8100TDXEN933 04/11/2019
15547587 8100TDXEN933 04/11/2019
15547589 8100TDXEN933 04/11/2019
15547590 8100TDXEN933 04/11/2019
15547591 8100TDXEN933 04/11/2019
15547593 8100TDXEN933 04/11/2019
15547597 8100TDXEN933 04/11/2019
15547598 8100TDXEN933 04/11/2019
15547600 8100TDXEN933 04/11/2019
15547602 8100TDXEN933 04/11/2019
15547604 8100TDXEN933 04/11/2019
15547605 8100TDXEN933 04/11/2019
15547606 8100TDXEN933 04/11/2019
15547607 8100TDXEN933 04/11/2019
15547608 8100TDXEN933 04/11/2019
15547609 8100TDXEN933 04/11/2019
15547611 8100TDXEN933 04/11/2019
15547612 8100TDXEN933 04/11/2019
15547613 8100TDXEN933 04/11/2019
15547614 8100TDXEN933 04/11/2019
15547615 8100TDXEN933 04/11/2019
15547616 8100TDXEN933 04/11/2019
15547617 8100TDXEN933 04/11/2019

15547618 8100TDXEN933 04/11/2019
15547619 8100TDXEN933 04/11/2019
15547620 8100TDXEN933 04/11/2019
15547621 8100TDXEN933 04/11/2019
15547622 8100TDXEN933 04/11/2019
15547623 8100TDXEN933 04/11/2019
15547624 8100TDXEN933 04/11/2019
15547625 8100TDXEN933 04/11/2019
15547626 8100TDXEN933 04/11/2019
15547627 8100TDXEN933 04/11/2019
15547628 8100TDXEN933 04/11/2019
15547629 8100TDXEN933 04/11/2019
15547630 8100TDXEN933 04/11/2019
15547631 8100TDXEN933 04/11/2019
15547632 8100TDXEN933 04/11/2019
15547633 8100TDXEN933 04/11/2019
15547634 8100TDXEN933 04/11/2019
15547635 8100TDXEN933 04/11/2019
15547636 8100TDXEN933 04/11/2019
15547637 8100TDXEN933 04/11/2019
15547638 8100TDXEN933 04/11/2019
15547639 8100TDXEN933 04/11/2019
15547640 8100TDXEN933 04/11/2019
15547642 8100TDXEN933 04/11/2019
15547643 8100TDXEN933 04/11/2019
15547644 8100TDXEN933 04/11/2019
15547645 8100TDXEN933 04/11/2019
15547646 8100TDXEN933 04/11/2019
15547647 8100TDXEN933 04/11/2019
15547648 8100TDXEN933 04/11/2019
15547650 8100TDXEN933 04/11/2019
15547652 8100TDXEN933 04/11/2019
15547653 8100TDXEN933 04/11/2019
15547655 8100TDXEN933 04/11/2019
15547656 8100TDXEN933 04/12/2019
15547657 8100TDXEN933 04/12/2019
15547658 8100TDXEN933 04/12/2019
15547659 8100TDXEN933 04/12/2019
15547660 8100TDXEN933 04/12/2019
15547661 8100TDXEN933 04/12/2019
15547665 8100TDXEN933 04/12/2019
15547666 8100TDXEN933 04/12/2019
15547668 8100TDXEN933 04/12/2019
15547669 8100TDXEN933 04/12/2019
15547670 8100TDXEN933 04/12/2019
15547678 8100TDXEN933 04/12/2019
15547679 8100TDXEN933 04/12/2019
15547680 8100TDXEN933 04/12/2019
15547682 8100TDXEN933 04/12/2019
15547684 8100TDXEN933 04/12/2019
15547685 8100TDXEN933 04/12/2019
15547687 8100TDXEN933 04/12/2019
15547688 8100TDXEN933 04/12/2019
15547689 8100TDXEN933 04/12/2019
15547690 8100TDXEN933 04/12/2019

15547692 8100TDXEN933 04/12/2019
15547694 8100TDXEN933 04/12/2019
15547695 8100TDXEN933 04/12/2019
15547697 8100TDXEN933 04/12/2019
15547726 8100TDXEN933 04/12/2019
15547734 8100TDXEN933 04/12/2019
15547745 8100TDXEN933 04/12/2019
15547753 8100TDXEN933 04/12/2019
15547759 8100TDXEN933 04/12/2019
15547761 8100TDXEN933 04/12/2019
15547767 8100TDXEN933 04/12/2019
15547770 8100TDXEN933 04/12/2019
15547771 8100TDXEN933 04/12/2019
15547774 8100TDXEN933 04/12/2019
15547777 8100TDXEN933 04/12/2019
15547778 8100TDXEN933 04/12/2019
15547781 8100TDXEN933 04/12/2019
15547783 8100TDXEN933 04/12/2019
15547784 8100TDXEN933 04/12/2019
15547791 8100TDXEN933 04/12/2019
15547793 8100TDXEN933 04/12/2019
15547795 8100TDXEN933 04/12/2019
15547798 8100TDXEN933 04/12/2019
15547799 8100TDXEN933 04/12/2019
15547808 8100TDXEN933 04/12/2019
15547829 8100TDXEN933 04/12/2019
15547837 8100TDXEN933 04/12/2019
15547839 8100TDXEN933 04/12/2019
15547840 8100TDXEN933 04/12/2019
15547842 8100TDXEN933 04/12/2019
15547847 8100TDXEN933 04/12/2019
15547850 8100TDXEN933 04/12/2019
15547867 8100TDXEN933 04/12/2019
15547869 8100TDXEN933 04/12/2019
15547896 8100TDXEN933 04/12/2019
15547897 8100TDXEN933 04/12/2019
15547902 8100TDXEN933 04/12/2019
15547904 8100TDXEN933 04/12/2019
15547908 8100TDXEN933 04/12/2019
15547911 8100TDXEN933 04/12/2019
15547916 8100TDXEN933 04/12/2019
15547918 8100TDXEN933 04/12/2019
15547930 8100TDXEN933 04/12/2019
15547936 8100TDXEN933 04/12/2019
15547948 8100TDXEN933 04/12/2019
15547950 8100TDXEN933 04/12/2019
15547957 8100TDXEN933 04/12/2019
15547959 8100TDXEN933 04/12/2019
15547964 8100TDXEN933 04/12/2019
15547969 8100TDXEN933 04/12/2019
15547975 8100TDXEN933 04/12/2019
15547977 8100TDXEN933 04/12/2019
15547979 8100TDXEN933 04/12/2019
15547981 8100TDXEN933 04/12/2019
15547983 8100TDXEN933 04/12/2019

15547989 8100TDXEN933 04/12/2019